



MEDIKIN™
Teaching Aides
BY LEGACY PRODUCTS INC.



P.O. Box 267 • Cambridge City, IN 47327 USA
800-238-7951 • Fax: 765-478-3266



Recommended Procedures for Infection Control & Cleaning

Infection control is important in medical institutions and environments around the world. For over 30 years Legacy Products has worked with major hospitals and clinics to design and produce patient education teaching materials for medical professionals and educators. All of our programs and products have been developed with their input and at their request.

For maximum infection control, we recommend the following:

- MediKins and adapters are tools designed for use by trained professionals. They should not be left unattended in common areas, playrooms or waiting rooms. When not in use, return them to their original plastic storage bags or other storage containers and place them in a secure location in an office or cabinet.
- Prior to and following the use of MediKin teaching tools, the educator/medical professional, patient and caregivers must wash their hands in the prescribed infection control manner.
- The materials used for all MediKins and adapters can be disinfected with clear sprays and wipes. It does not harm the fabric to be dampened by clear disinfectants used on other fabrics in your facility. After disinfecting, make sure all products are completely dry before returning them to their storage bags/containers.
- When using the MediKin teaching aides, encourage the patient, educator/medical professionals and caregivers to also "play doctor" by wearing masks and gloves throughout the teaching process.
- MediKins and adapters can be cleaned with clear fabric/upholstery cleaners. Follow the manufacturer's instructions and test the product in a small hidden area first. After cleaning, make sure all products are completely dry before returning them to their storage bags/containers.

Roughly 75% of our orders each year are re-orders from prestigious institutions that have used our products through the years. They have met their facility's stipulations for infection control by following the above procedures.